PRODUKTEMPFEHLUNG | PRODUCT
RECOMMENDATION | RECOMMANDATION
PRODUIT | RACCOMANDAZIONE DEL
PRODOTTO

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Respiratory physiotherapy apparatus
Moisturizing, mucolytic and regenerating.

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EN RC-RHINO HORN™ opens your nasal
passages when you have a cold, sinusitis, a
pollen or dust allergy, or a snoring problem;
when you are pregnant; and cleans your nasal
cavity and passages after surgery.

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www.cegla-shop.com

MADE OF RECYCLABLE PEHD – NO PHTHALATES
NO BISPHENOL A (BPA) – TOXIC FREE ACCORDING TO REACH 168
TEST CE MARKED IN CONFORMITY WITH EU DIRECTIVE 93/42/EEC

REF 821
PZN 124 360 04
PPN 111 243 360 047 5

CE

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Manufactured under
ISO 13485:2003 by:
Yogaprocess AS
P. O. Box 2599 Solli,
N-0203 Oslo, Norway

Rhino Horn™ is a registered trade-
mark owned by Yogaprocess AS

Vertrieb | Sales |
Distribué par |
Distribuzione:
R. Cegla GmbH & Co. KG
Horresser Berg 1
D - 56410 Montabaur
www.cegla.de

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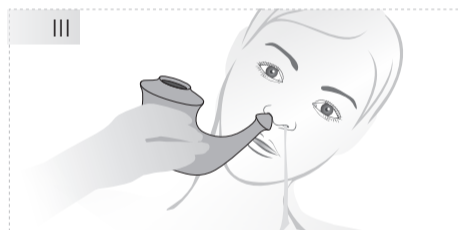
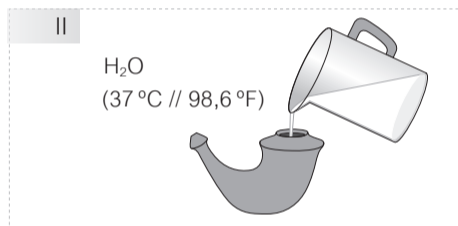
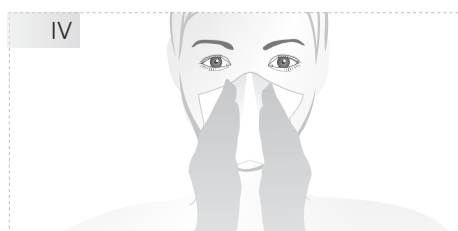
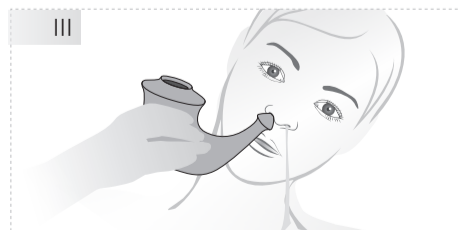
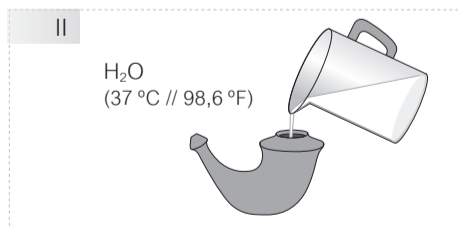
USER INSTRUCTIONS

EN USER INSTRUCTIONS

Before using the RC-Rhino Horn™ for the first
time, you should rinse it thoroughly with running
water.

THE WATER

Prepare a saline solution by filling the RC-Rhino
Horn™ with a spoonful of ordinary kitchen salt
(2,7 g NaCl) and lukewarm water (37 °C / 98.6 °F).
Use the measuring spoon included with the RC-
Rhino Horn™ for the correct amount of salt. Close
the openings of the RC-Rhino Horn™ with your
palm / finger and shake it a few times to dissolve
the salt. If there are concerns about water quality,
boil the water for one minute and then let it cool
to body temperature. If the tap water is safe, you
may use it directly from the warm tap.

RINSING THE NOSE

To use the RC-Rhino Horn™, tilt your head side-
ways over the sink and from the side gently
place the spout into the upper nostril. Start with
your less congested nostril. Breathing normally
through your open mouth, pour the saline solution
into your upper nostril, and if the angle of the head
is correct, the water will drain directly through the
lower nostril. Do not stick your chin out; keep it

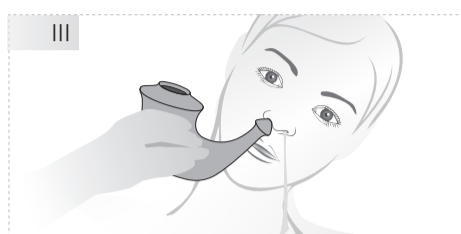
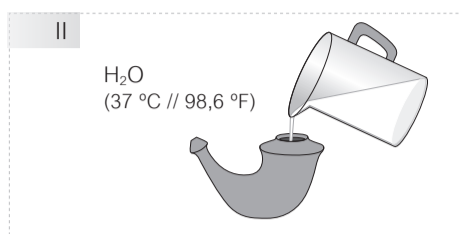
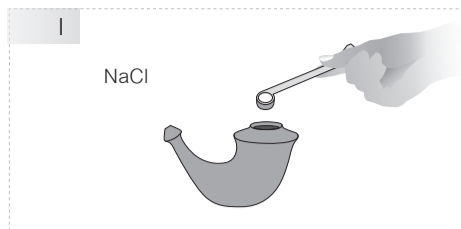
tucked in towards your chest, otherwise the water
will run back from the nasal cavity into your throat.
Repeat on the other side.

EMPTY YOUR NOSE

To clear any remaining water from your nose and
sinuses after you use the RC-Rhino Horn™, bend
forward with the top of your head down, look to
the side, close the lower nostril and gently blow
into a tissue through the open nostril. Be careful
not to blow so hard that your ears pop. Repeat on
the other side. Do this a few times.

CLEANING THE RC-RHINO HORN™

After each use, the RC-Rhino Horn™ should be
rinsed out and left to air-dry. Virus and bacte-
ria do not survive long on a dry plastic surface.
The RC-Rhino Horn™ may be washed in a dish-
washer and may even be cleaned in boiling water
for a few minutes. Washing or boiling is necessary
only if two persons are using the same RC-Rhino
Horn™ right after one another. Normally air-drying
is enough.

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Respiratory Physiotherapy Device
Most effective for secretion mobilization, reducing chronic cough and breathlessness.

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Nasal Moisturizing Gel for dry nose
Provides the nasal mucous membranes with natural moisture when the nose is dry.

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Nose and Pharynx Spray
Protects and cares for nose and pharynx.

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Reusable microwaveable cleaning bag.
Easy and quick cleaning of medical products as well as baby items. Use only for microwave safe products.

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